

Relevant information about the BreaSAFE® ANTI-COVID-19 respirator

Dear users,

you have just purchased respirator of a new type BreaSAFE® ANTI-COVID-19. This respirator contains an antibacterial nanofibrous component. The effects of the respirator have been tested by an accredited testing laboratory of the Occupational Safety Research Institute and have been approved by The Czech Trade Inspection Authority. Based on these documents, the product has been granted the FDA approval for introduction onto the American market.

Respirator BreaSAFE® ANTI-COVID-19 is intended for protection of the user against the SARS-CoV-2 virus which causes the COVID-19 disease. In compliance with the requirements of the World Health Organization (WHO) for this particular application, the nominal protective factor given by this filtrating half-mask does not differ from the nominal protective factor FFP2 defined in EN 149: 2001 + A1: 2009. This filtrating half-mask is not intended for general use and must not be used for other purposes than as a protective aid against COVID-19.

This respirator does not protect against gases. It must not be used for protection in environments where the composition and concentration of air-borne pollutants is unknown.

This respirator is designed to be reusable. It is not to be washed, and in order to minimize the risk of possible microbial contamination, its surface areas can be treated with spray-application of a disinfectant agent after the use.

The lifetime of the respirator depends on an environment in which the user wears it and also on the activity of the user.

The respirator must not be immersed in water or any other liquid.

Warning – this product is not fire-resistant and must not be used near open fire.

The respirator must be replaced if:

- Dust has penetrated and polluted the respirator
- Breathing resistance has significantly increased
- Respirator has been mechanically damaged
- Apparent contamination has occurred

Prior to use, please assure that the respirator is not damaged. Follow the instructions when placing the respirator on your face:



1. Take the respirator and place it on your face with the nose part pointing upward.



2. Pull the straps over your head. Adjust the length of the straps to ensure that the respirator does not move on your face but also that it will leave no press marks.



3. Check proper sealing of the respirator: Inhale through it forcefully. If you feel that air is leaking between the respirator and your face, you will achieve better sealing by:



Phone: +420 565 552 610
E-mail: breasafe@pardam.cz
www.pardam.cz

PARDAM
NANO4FIBERS s.r.o.
ID No.: 25268694
VAT No.: CZ25268694

Headquarters:
Žižkova 2759
413 01 Roudnice n/L
Czech Republic

Fio Banka, a.s.
(BIC) SWIFT: FIOBCZPPXXX
CZK – account no.: 2501552451/2010
IBAN: CZ 562010000002501552451

EUR: account no.: 2801552453/2010
IBAN: CZ372010000002801552453

- repositioning the respirator to a more suitable position
- adjusting the length of the straps
- repositioning the straps to a more suitable position

Sealing requirements will probably not be met if the user has beard in the area of the sealing line of the respirator.



Phone: +420 565 552 610
E-mail: breasafe@pardam.cz
www.pardam.cz

PARDAM
NANO4FIBERS s.r.o.
ID No.: 25268694
VAT No.: CZ25268694

Headquarters:
Žižkova 2759
413 01 Roudnice n/L
Czech Republic

Fio Banka, a.s.
(BIC) SWIFT: FIOBCZPPXXX
CZK – account no.: 2501552451/2010
IBAN: CZ 5620100000002501552451

EUR: account no.: 2801552453/2010
IBAN: CZ3720100000002801552453